

NJ Phillips Metal Automatic Injector 5ml

(PAS1198) (PAS265 with Bottle Attachment)



MODEL: PAS1198
NEW MODEL AVAILABLE WHEN
CURRENT STOCK EXHAUSTED

MODEL: PAS265
NEW MODEL AVAILABLE WHEN
CURRENT STOCK EXHAUSTED

Suitable for: Cattle, Pigs & Sheep

- Robust and highly accurate design.
- Administers a variety of vaccines.
- Calibrated from 0.5ml to 5ml.
- Superior comfort push grip and handle.

SPARE PARTS – PLEASE ORDER BY KIT & PART NAME:

KIT No.	PART NAME	PART No.
WX111	Major Service Kit	5,6,7,10,11,12,14.
WX1061	Minor Service Kit	5,6,7,11,12,14.
WX610	A, D & E Kit	Not shown.

LEGEND PAS1198:


1. Needle Nut (SN45/N)
2. Cylinder Cover (SN564)
3. Finger Grip (SP1316)
4. Barrel Lock Ring (SN565)
5. Needle Mount Assembly (SXM28)
6. Head Seal Washer (SH45A)
7. Delivery Valve Washer (SF4)
8. Cup Support Washer (SN56)
9. Delivery Valve Retaining Screw (SN54)
10. Cylinder (SH1502)
11. Inlet Valve Washer (SF4)
12. Cup Support Washer (SN56)
13. Piston (SN113)
14. Piston Seal Ring (SP323)
15. Cylinder Lock Nut (SN74)
16. Dose Adjustor Lock Nut (SN73)
17. Dose Adjustor (SN72)
18. Return Spring (SP145)
19. Spring Cover (AN199)
20. Push Rod & Grip (SXM30A)

SPARE PARTS – PLEASE ORDER BY KIT & PART NAME:

KIT No.	PART NAME	PART No.
WX1778	Major Service Kit 5ml	3,4,5,12,13,14,15,23,24,25,26,28,30,31.
WX1777	Minor Service Kit 5ml	3,4,5,12,13,15,23,24,25,26,28,30,31.
WX1775	Major Kit - Quick Release Bottle Attachment	23,24,25,26,27,28,29,30,31.
WX1774	Minor Kit - Quick Release Bottle Attachment	23,24,25,26,28,29,30,31.
WX1781	Conversion Kit (Feedtube Attachment Fitting)	Not Illustrated.

LEGEND PAS265:

1. Needle Nut (SN45/N)
2. Needle Mount (SN373/N)
3. Delivery Valve Spring (SP41)
4. Delivery Valve & Seal Ring (WX1270)
5. Needle Mount Seal Ring (SP66)
6. Delivery Cage Assembly (SXM633)
7. Delivery Cage Seal Ring (SP1)
8. Cylinder Cover (SN564)
9. Handle (SP1316)
10. Cylinder Lock Nut (SN565)
11. Head Seal Fitting (SN165)
12. Head Seal Washer (SH45A)
13. Cylinder Seal Ring (SP176)
14. Cylinder (SH1502)
15. Piston Seal Ring (SP323)
16. Piston (SN164)
17. Lock Nut (SN74)
18. Dose Adjustor Lock Nut (SN73)
19. Dose Adjustor (SN72)
20. Return Spring (SP145)
21. Spring Cover Plate (AN199)
22. Push Rod Assembly (SXM599)
23. Draw-Off Needle Seal Ring (SP309)
24. Inlet Valve Spring (SP1472)
25. Inlet Valve (SH131)
26. Inlet Valve Seal Ring (SP631)
27. Bottle Retaining Shroud (SH1643, SH1666, SH1667)
28. Air Bleed Valve (SP916)
29. Draw-Off Needle (SH1642)
30. Needle Shroud (SH1663)
31. Spanner (SM800)

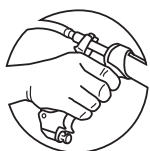

njphillips.com

Our expertise is in your hands.

NJ PHILLIPS PTY LIMITED ABN 36 000 082 002

44 Gindurra Road, Somersby NSW, Australia 2250

Postal Address: Locked Bag 8, Gosford NSW, Australia 2250

Phone: +61 2 4340 2044 **Fax:** +61 2 4340 1991

Email: njp1@njphillips.com.au



The information contained in this document is legally privileged and confidential intended only for the use of the individual of entity. If the receiver of this document is not the intended recipient, the receiver is hereby notified that any dissemination, distribution or copy of this information is strictly prohibited. If this information is received in error, please notify us immediately. As our policy is one of continuous improvement, the manufacturer reserves the right to alter these specifications at any time. All products produced by NJ Phillips Pty Limited are identified by a unique batch number which is affixed to the product to allow traceability by the manufacturer and must not be removed if product integrity is to be maintained. Not all part numbers may be available.